Swiss Point of Care

Lancets Package Insert


REF G124-90AA	English
---------------	---------

PRINCIPLE AND INTENDED USE

The *Swiss Point of Care* Lancets are used with the *Swiss Point of Care* Lancing Device to obtain small amounts of blood samples. For self testing and professional use.

INSTRUCTION FOR USE

- Step 1. Prior to testing, wipe the test site with an alcohol swab or soapy water. Then dry your hands and the test site thoroughly. Make sure there is no cream or lotion on the test site.
- Step 2. Insert a sterile lancet into the lancing device and push it until the lancet comes to a complete stop in the lancing device.
- Step 3. Hold the lancet firmly in the lancing device and twist the safety tab until it loosens, then pull the safety tab off the lancet. Save the safety tab for lancet disposal.
- Step 4. Obtain the required blood sample with the sterile lancet and lancing device.
- Step 5. Place the safety tab of the lancet on a hard surface and carefully insert the lancet needle into the safety tab. Then discard the used lancet.

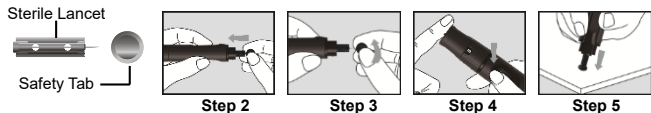See the *Swiss Point of Care* Lancing Device insert for more details to obtain blood samples.

PRECAUTIONS

- Do not use the lancet if the safety tab is missing or loose when you take the lancet out of the bag.
- Do not use the lancet if the needle is bent.
- Use caution whenever the lancet needle is exposed.
- Never share lancets with other people.
- Always use a new, sterile lancet to reduce the risk of infection from prior use of the instrument. Do not reuse lancets.
- Avoid getting the lancets dirty with hand lotion, oils, dirt or debris.

Index of Symbols

	Consult instructions for use	LOT	Lot number	REF	Catalogue number
	Manufacturer		Use by		Do not reuse
STERILE R	Sterilized using irradiation	EC REP	Authorized representative in the European Community		

ACON Laboratories, Inc.
5850 Oberlin Drive, #340
San Diego, CA 92121, USA
www.aconlabs.com

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MDSS GmbH