2211-TITAN

301 / 304 / 304F / 308

Electric Treatment Table / Camilla de Tratamiento Eléctrica / Table de Traitement
Électrique / Lettino Elettrico / Elektrischer Behandlungstisch

EN	3
ES	7
FR	12
IT	16
DE	20

INTRODUCTION

DDUUEETT Europe SL thanks you for choosing this product. We hope this instruction manual will help you to get the most out of this product.

Contact your dealer if you need clarification regarding its use or operation.

Read the entire instruction manual carefully before using the table. Take note of the warnings in this manual.

Always keep this instruction manual and should you give the table to another operator make sure you also provide the manual since it is an essential part of the product.

Some table parts may differ from the image that appears in catalogues, the web, trade brochures or other publications.

In order to make improvements to the table, the manufacturer reserves the right to modify its characteristics or parts without prior notice.

If, during the useful life of this table, any accident occurs that causes or could have caused serious injuries to the user, DDUUEETT Europe SL must be informed immediately.

WHAT'S IN THE BOX

Electric treatment table: 1 unit.

Hand controller: 1 unit. (mod. 2211-304).

Power cable: 1 pc.

Instruction manual: 1 pc.

Paper roll holder: 1 pc.

REGULATION AND PRODUCT STANDARDS

This table complies with the European regulation of medical devices (EU) 2017/745 and with the following standards:

- EN 60601-1:2006/A2:2021
Medical electrical equipment. Part 1: General requirements for basic safety and essential performance.
- EN 60601-1-6:2010/A2:2021
Medical electrical equipment. Part 1-6: General Requirements for basic safety and essential performance. Collateral standard: Suitability for use.
- EN 60601-1-2:2015
Medical electrical equipment. Part 1-2: General requirements for basic safety and essential performance. Collateral standard: Electromagnetic disturbances. Requirements and tests.

SAFETY SYSTEM

The electric treatment tables can be optionally equipped with the integrated safety system to comply

with the German standard of the Federal Institute for Drugs and Medical Devices (BfArM), (FRG).

The electrical height adjustment of our tables is activated only after a triple press in quick succession of the perimeter bar or foot switch, if available, to prevent unintentional or unauthorized adjustment.

Once the height adjustment is completed, a correction can be made within 5 seconds before the bed automatically shuts down. If the interruption lasts longer than 5 seconds the system must be pressed again three times for activation.

CORRECT USE OF THE PRODUCT

This table is designed for use in clinical, aesthetic and rehabilitation settings, including clinics, physiotherapy centres, care homes and doctors' surgeries.

This table is intended to support users during treatment and should not be used to transport patients.

Do not make any changes to the table without DDUUEETT Europe SL's authorisation.

Please note that any misuse or modification of the table voids the warranty.

Contraindications: Not discovered yet.

PRECAUTIONS AND SAFETY INDICATIONS

DDUUEETT Europe SL is not liable for any material or personal damage due to misuse or incorrect use of this table for any purpose other than those described in the "CORRECT USE OF THE PRODUCT" section above.

Inspect the good condition of the table before using it for the first time and on a recurring basis as indicated in the "MAINTENANCE" section.

Do not use the table outdoors or in places where it may be affected by moisture or adverse weather conditions.

The ideal conditions for use are a temperature between 5-40 °C, a relative humidity between 30-75% and atmospheric pressure between 860-1060 hPa.

Do not use the table in environments where other devices may emit electromagnetic radiation. These radiations could affect the correct functioning of the table.

This table can generate electromagnetic energy that can interfere with other nearby electronic devices.

The table should be kept out of the reach of children when there is no adult supervision.

The table is designed for discontinuous interval operation, so the continuous running time of the motor should never exceed 2 minutes. Should this happen, damage may occur due to the overheating of the electronics.

Always use original accessories and spare parts provided by your dealer.

Always cover the table with a towel, disposable paper or a cover and change it for each user, thus ensuring correct hygiene and extending the useful life of the upholstery.

Only connect the table to a properly installed. The local mains voltage must correspond to the technical data of the device.

The power outlet should be accessible in case the table needs to be quickly unplugged at any given time.

Always pull the plug and not the mains cable.

Unplug the table from the power supply before starting cleaning or any type of maintenance operations.

If the electrical supply cable is defective, disconnect the table immediately and inform your dealer or a specialised technician.

Position the power cord properly so that it cannot be tripped over.

Unplug the power cord from the mains if the table is not going to be used for a long period of time.

If you detect any abnormality in the operation of the table, consult your dealer or a specialised technician.

Never manipulate any electrical part or open or repair any part of the table.

Only your dealer or a specialised technician should repair the table.

Never touch the table or the power cord with wet hands.

Never place your hands or any other part of your body in any hole or void on the table while it is in operation.

Make sure there are no objects in the space under the table that could cause collisions or entrapment during the movement of the same.

Make sure that your feet are not resting on the peripheral foot bar during treatment and especially when sitting near the table. This could create a dangerous situation where your leg is caught between the section and the peripheral foot bar.

Take into account the weight distribution on the table when working. The weight must be distributed along the table without concentrating on its ends to maintain the correct stability of the same.

You should not sit on the head and foot sections. If you do, there is a danger of tipping.

INSTALLATION

Put the product to a proper location, according to the overall layout, the daylighting or artificial lightings and operation condition of the salon's or the home. And keep it in the clean, dry and cool environment.


Open the packing carton to check if the product is intact and whether the accessories or spares are complete. Please contact distributor or us if there's any problem.

Place the product to the proper position. Please note that it is unnecessary to install extra screw on the


floor to fasten the bed, because the original design has taken the stability into consideration.

But please ensure the product is placed on a firm and flat ground, so as to prevent any accident. If the product rotates or slant due to rough ground, please adjust the four pieces of plastic feet (M10) and nuts manually or using a hexagon open-end wrench, which are fasten on the bottom panel, so as to ensure the product is stable.

Connect the plug of product with power supply properly. The table is equipped with a three-wire power cord and must be connected. This connection is essential to avoid electrical hazards and to ensure proper operation.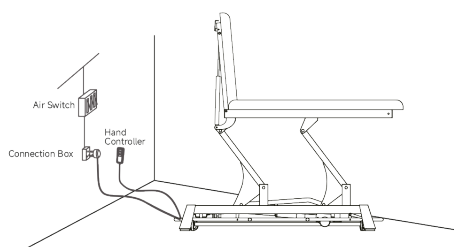

Place the paper roll holder in the position indicated in the image below, inserting it into the two holes in the structure of the treatment table.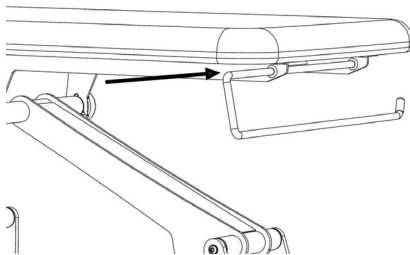

PLACEMENT IN THE WORKPLACE


For proper use, the table must be placed on flat, stable ground. Floor anchors are not required.

If you need to move the table, do so by holding it by the base, never by the upholstery or the cushions. The table has 4 wheels to facilitate its movement to the place where it will be used.


Once correctly positioned, the table has an adjustable immobilisation system that prevents it from moving during use.

To activate or deactivate this system, you must press either of the two levers with your foot, as shown in the following images.

- Image 1: Table in working position.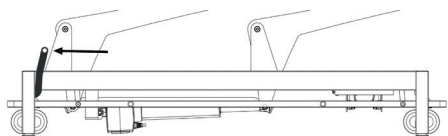

- Image 2: Table in position for movement.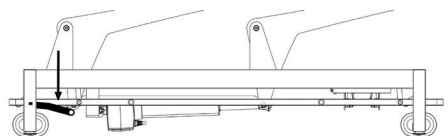



OPERATIONS

It can be operated via hand controller, foot controller, or perimeter controller.

Motor functions for perimeter controller.

- General raising/lowering of the table
 - Use the manual controller or the integrated foot controller (depending on the model). Press and hold the corresponding button until the desired position is reached.
 - Do not exceed 2 minutes of continuous operation. Wait at least 18 minutes before operating the motor again.

1. Standard Configuration (mod. 2211)
(Perimeter Controller)

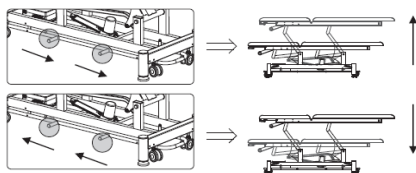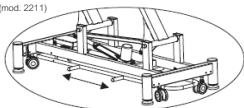Manual functions

- Backrest tilt
- Legrest tilt (depending on the model)

See page 24.

PRODUCT INFORMATION

From page 24 you can find detailed information: sketch of the table, measurements and possible angles of movement of each section.

LEGEND TO SYMBOLS AND INFORMATION ON THE PRODUCT LABEL.

- **MODEL** : Product identification.
- **CE** : CE conformity marking.
- **SN** : Unique serial number identifying the product.
- **W** : Date of manufacture.
- **INPUT**: Necessary values of the electrical mains supply.
- **ON: 2 min / OFF: 18 min**: Indicates the discontinuous operation of the device according to IEC/EN 60601-1, which indicates that 2 minutes of continuous operation of the motor or motors should not be exceeded.
- **Consult the instructions for use or the user manual.**
- **Electrical Safety Classification - Type B.**
- **IPX4**: Protection against splashing water or liquids.
- **Manufacturer identification.**
- **EU REP**: Authorised representative in the EU.
- **Importer.**
- **MD**: Medical device.
- **UDI**: Unique device identifier.
- **The product should not be discarded as unsorted waste.**

TROUBLESHOOTING

If, during the life of the table, you detect any malfunction or need a repair, contact your dealer and do not use the table until you have received a favourable diagnosis or the table has been repaired.

MAINTENANCE

Carry out a visual inspection at least once a month to check the general conditions of the table. Moreover, you must do the following:

- Check the correct adjustment of the different sections.
- Check the proper functioning of the motors and their movements.
- Check and clean the gaslifts.

- Check and tighten screws (if necessary).
- Grease the parts with friction.
- Check the conditions of the electrical wiring and the socket.
- Check the wheel immobilisation system.

USEFUL LIFE OF THE TABLE

The table has a useful life of 8 years. However, this period can be extended for periods of 2 years, according to assessments made by an authorised distributor or DDUUEETT Europe, SL.

RECYCLING

Destroy and dispose of this product safely following your community's recommendations for the disposal of electronic products. Please be mindful of environmental concerns and never dispose of motors or electrical parts in the municipal trash. If you suspect the product may be biologically contaminated, use your community's disposal procedures.

WARRANTY

DDUUEETT Europe SL guarantees the products for manufacturing defects for 2 years from the invoice date.

Any warranty will be void if the instructions for use and maintenance have not been observed, if changes have been made to the products, if parts have been replaced, or if materials that do not meet the specifications of the original manufacturer's products have been used.

CLEANING AND DISINFECTION

Before cleaning the table, unplug it from the mains.

Clean the table after each use.

Dust the table with a soft, dry cloth.

To clean the upholstery, apply a solution of warm water and a mild detergent (for example, neutral soap) with a well wrung soft cloth.

Disinfect upholstery and other surfaces with a suitable bactericide for this purpose.

Apply the disinfectant on a soft cloth, never directly on the table.

Never use a mixture of disinfectants or a mixture of disinfectant and soap.

Avoid the accumulation of moisture on the moving parts of the metal structure, as they could rust.

Since we do not know which disinfectant will be used, we cannot guarantee that it will not cause damage to the upholstery. Read the instructions and warnings for use of the disinfectant product carefully before its purchase or use.

CONTACT

Importer:

DDUUEETT Europe, SL
Font de la Ventaïola, 1-3 Nau 3
08670, Navàs
Barcelona, España
info@dduueett.com
0034 93 737 81 02

European legal representative:

Lotus NL B.V.
Koningin Julianaplein 10, 1e Verd
2595AA, The Hague, Netherlands
Tel: +31644168999
E-mail: info@lotusnl.com

Manufacturer:

SILVERFOX CORPORATION LIMITED
No. 18, 1st TongLe Road, TangXia Town,
PengJiang District, JiangMen City,
GuangDong Province, P.R. China

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